CaviCide™

Disinfectant solution for surfaces of non-invasive medical devices

(es) Solución desinfectante para superficies de dispositivos médicos no invasivos (fr) Solution désinfectante pour les surfaces d'appareils médicaux non invasifs (it) Soluzione disinfettante per superfici di dispositivi medici non invasivi (tr) İnvazif olmayan tıbbi cihazların yüzeylerine yönelik dezenfektan çözelti (nl) Desinfecterende oplossing voor oppervlakken van niet-invasieve medische instrumenten (pt) Desinfetante para superfícies de dispositivos médicos não invasivos (de) Desinfektionsmittel für Oberflächen von nicht-invasiven medizinischen Vorrichtungen (en) Disinfectant solution for surfaces of non-invasive medical devices (ru) Раствор для обеззараживания поверхностей неинвазивных медицинских изделий (sk) Dezinfekčný roztok na povrchy neinvazívnych zdravotníckych pomôcok (da) Desinfektionsmiddel til overflader på ikke-invasivt medicinsk udstyr (no) Desinfiserende løsning for overflater på ikke-invasivt medisinsk utstyr (pl) Dezynfekujący roztwór do powierzchni nieinwazyjnych wyrobów medycznych (hu) Fertőtlenítő oldat nem invazív orvostechnikai eszközök felületéhez (ro) Dezinfectiionen razтвор za povrşnosti na neinvazivni medicinski izdelia (bg) Разтвор за дезинфекция на повърхности на неинвазивни медицински изделия (hr) Solucije dezinfektantâ pentru suprafeţele dispozitivelor medicale neinvazive (el) Απολυμαντικό διάλυμα για επιφανείες μη επεμβατικών ιατρικών συσκευών (ar) محلول مطهر لأسطح الأجهزة الطبية غير الباضعة

(fr) Solution désinfectante pour les surfaces d'appareils médicaux non invasifs (it) Soluzione disinfettante per superfici di dispositivi medici non invasivi (tr) İnvazif olmayan tıbbi cihazların yüzeylerine yönelik dezenfektan çözelti (nl) Desinfecterende oplossing voor oppervlakken van niet-invasieve medische instrumenten (pt) Desinfetante para superfícies de dispositivos médicos não invasivos (de) Desinfektionsmittel für Oberflächen von nicht-invasiven medizinischen Vorrichtungen (en) Disinfectant solution for surfaces of non-invasive medical devices (ru) Раствор для обеззараживания поверхностей неинвазивных медицинских изделий (sk) Dezinfekčný roztok na povrchy neinvazívnych zdravotníckych pomôcok (da) Desinfektionsmiddel til overflader på ikke-invasivt medicinsk udstyr (no) Desinfiserende løsning for overflater på ikke-invasivt medisinsk utstyr (pl) Dezynfekujący roztwór do powierzchni nieinwazyjnych wyrobów medycznych (hu) Fertőtlenítő oldat nem invazív orvostechnikai eszközök felületéhez (ro) Dezinfectiionen razтвор za povrşnosti na neinvazivni medicinski izdelia (bg) Разтвор за дезинфекция на повърхности на неинвазивни медицински изделия (hr) Solucije dezinfektantâ pentru suprafeţele dispozitivelor medicale neinvazive (el) Απολυμαντικό διάλυμα για επιφανείες μη επεμβατικών ιατρικών συσκευών (ar) محلول مطهر لأسطح الأجهزة الطبية غير الباضعة

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* If used as a disinfectant spray, the contact time is 1 minute.
If used as an immersion disinfectant solution, the contact time is 3 minutes.

Use-by date  SEE CONTAINER

ORGANISM

<i>Mycobacterium tuberculosis</i> var: bovis (BCG)	3
Methicillin Resistant <i>Staphylococcus aureus</i> (MRSA)	3
<i>Pseudomonas aeruginosa</i>	1
<i>Salmonella enterica</i>	3
<i>Staphylococcus aureus</i>	1
Vancomycin Resistant <i>Enterococcus faecalis</i> (VRE)	2
<i>Staphylococcus aureus</i> with reduced susceptibility to vancomycin	2
<i>Enterococcus hirae</i>	1
<i>Escherichia coli</i>	1
<i>Proteus mirabilis</i>	1
Human Coronavirus	2
Hepatitis B Virus (HBV)	2
Hepatitis C Virus (HCV)	2
Human Immunodeficiency Virus (HIV-1)	2
Herpes Simplex Virus Types 1 and 2	2
Influenza A2 Virus	2
Vaccinia Virus Ankara (MVA)	5
<i>Candida albicans</i>	1*
<i>Trichophyton mentagrophytes</i>	3

*CONTACT TIME (MINUTES)

CE
0044

25°C
5°C

Rx only

2673207r17
2020-10-16

SpofaDental a.s.
Markova 238,
506 01 Jicin
Czech Republic,
www.spofadental.com
+420 493 583 111
Made in Czech Republic

Disinfectant solution for surfaces of non-invasive medical devices. Tuberculocidal, Bactericidal, Virucidal, Fungicidal, and Yeastocidal.

Directions for use: For professional use only. Do not use if seal is broken. See container for lot number and expiration date.

For use as a disinfectant on non-invasive medical device

surfaces: Spray CaviCide directly onto pre-cleaned medical device surfaces, thoroughly wetting area to be disinfected. Allow medical device surfaces to remain visibly wet for required contact time at room temperature. Wipe surface dry using a towel or allow surface to remain undisturbed until visibly dry.

For use as a disinfectant soak for pre-cleaned, non-invasive medical devices and implements:

Medical devices must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow medical devices. Using either a tray or an ultrasonic unit, immerse medical devices in CaviCide for required contact time at room temperature. Remove and rinse medical devices. Wipe dry prior to use. Discard solution after each use.

Storage and disposal: Store in a well-ventilated place. Keep cool. Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation. Discard 90 days after opening.

Warning: Causes serious eye irritation. Keep away from heat, hot surfaces, open flames, sparks. No smoking. Keep container tightly closed. Wear eye protection, protective gloves. If eye irritation persists: Get medical advice/attention.

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