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Rescue box for tooth accidents

Intended purpose

Tooth rescue box to temporarily store and transport broken teeth or tooth fragments until replantation.

In the event of a dental accident:

1. Find and have ready the tooth or tooth fragment that has been knocked out. **Caution: Do not touch the tooth at the root or fracture surface! Do not clean or disinfect the tooth!**
2. The SOS Zahnbox is ready for immediate use. Before opening check expiry date and colour.
3. Unscrew the closure of the box just before use and immediately put the knocked-out tooth or broken tooth into the Zahnbox.
4. Close the SOS Zahnbox and note the patient's name, date and time on the bottle label.
5. Immediately go to a dental practice or dental clinic.

Information for the treating dentist:

Prior to its replantation, the tooth needs to be rinsed thoroughly with the NaCl solution. The solution in the **SOS Zahnbox** keeps the cells at the tooth root viable for up to 48 hours at room temperature. The chances for a successful replantation increase if the tooth or the tooth fragment is transferred into a new **SOS Zahnbox** after 24 hours.

Application:

Nutrient solution for the temporary storage of

- avulsed teeth – which are to be replanted
- extracted teeth – which are to be replanted or transplanted
- tooth fragments – which are to be reattached

Caution:

The miradent **SOS Zahnbox** contains a ready-for-use solution. If this solution turns turbid and changes colour from light red to yellow, the liquid is contaminated and must no longer be used. This product is for single use only and

cannot be sterilized after opening. Re-use will mean sterility is no longer guaranteed and can result in a high risk of infection.
Keep out of reach of children.

Store in a dry place away from light.

Storage temperature: 4 °C to 25 °C

Not for internal use. Do not open prior to use!

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